


# IntelViewer for MacOS 5.3.1


1. Download IntelViewer from this link:

<https://www.dropbox.com/s/iwjzdjn45asn2gk/IntelViewer%205-3-1-R8.dmg?dl=1>


2. Click **ALLOW** on the pop up

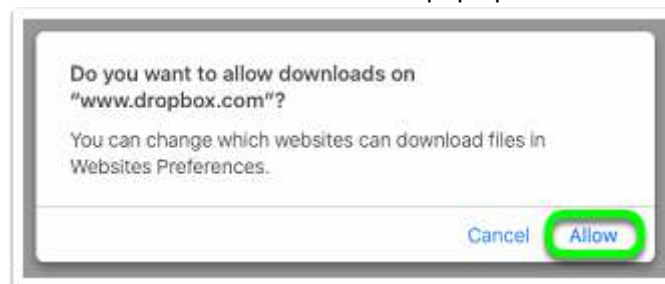


3. Click on IntelViewer 5.3.1

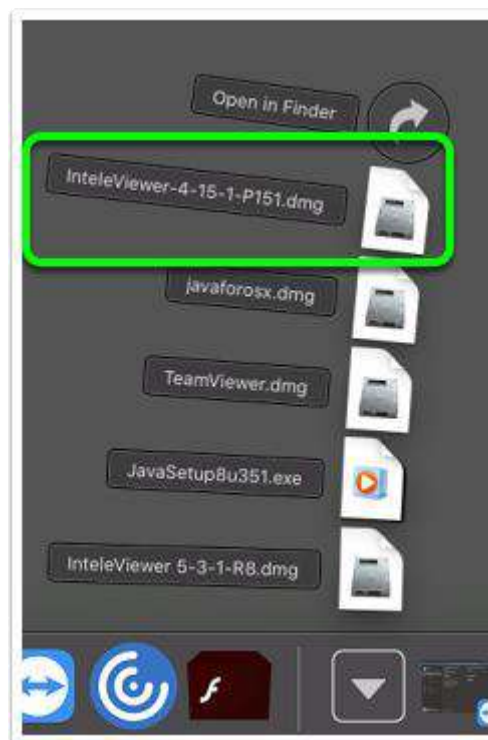


4. Drag IntelViewer to the applications folder

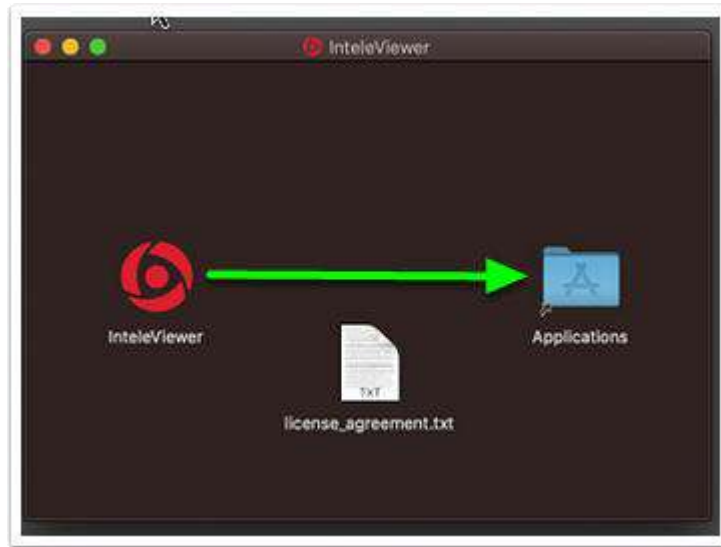


5. Launch IntelViewer from the Applications folder

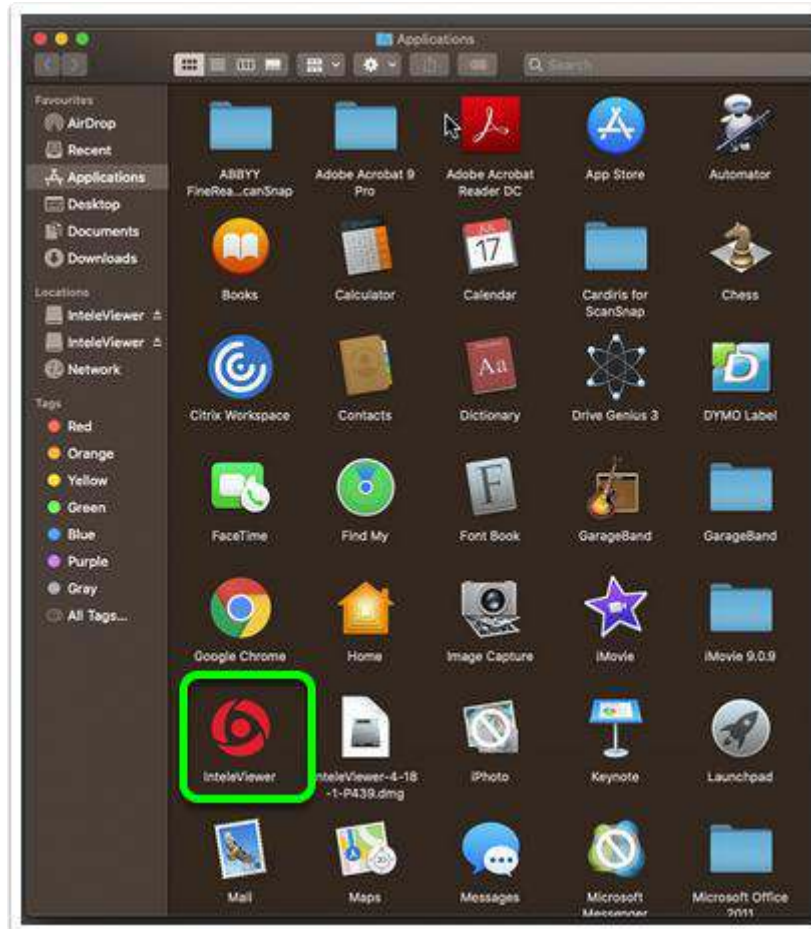


6. You will get the below security prompt – click **CANCEL**:

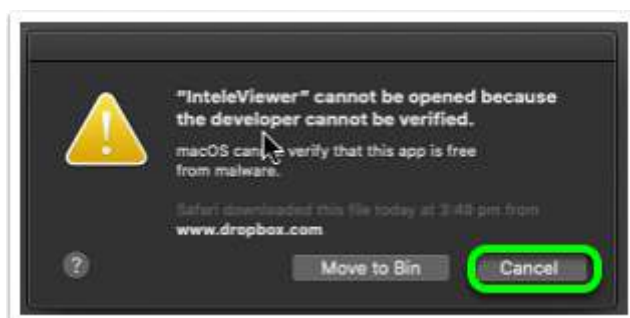


7. You will then need to go to System Preferences > Security & Privacy

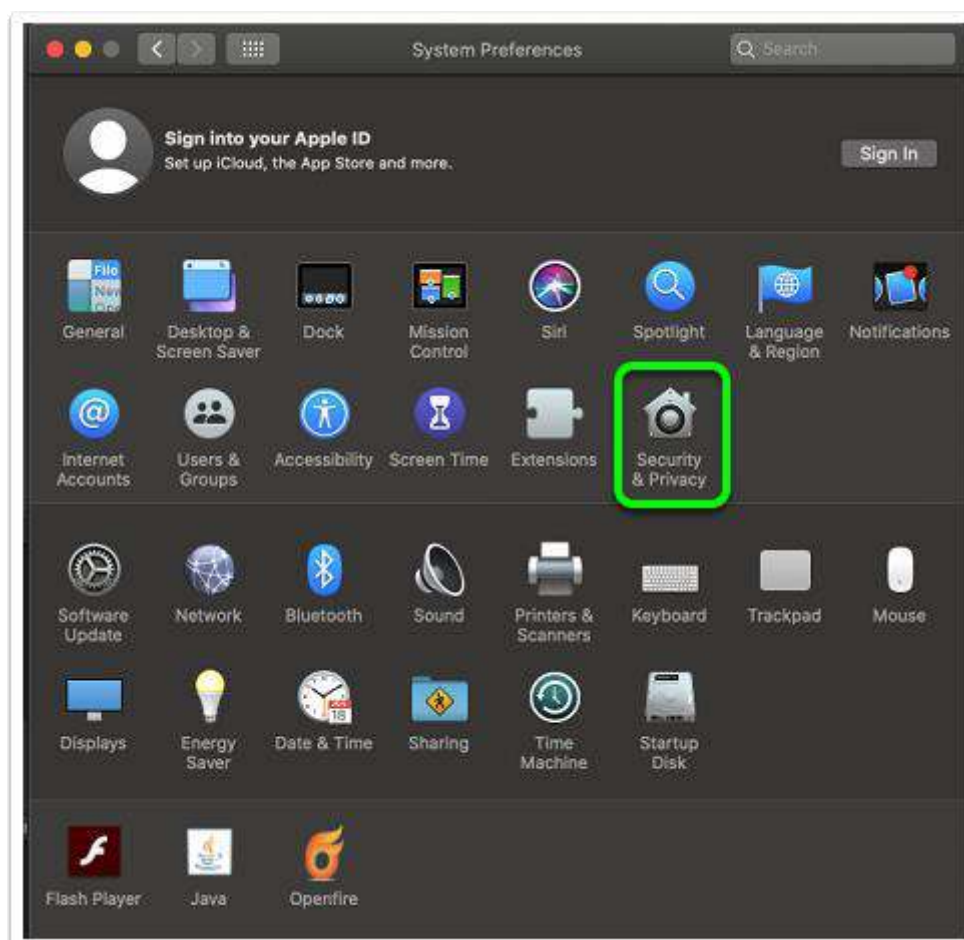

8. Click on **OPEN ANYWAY** for IntelViewer

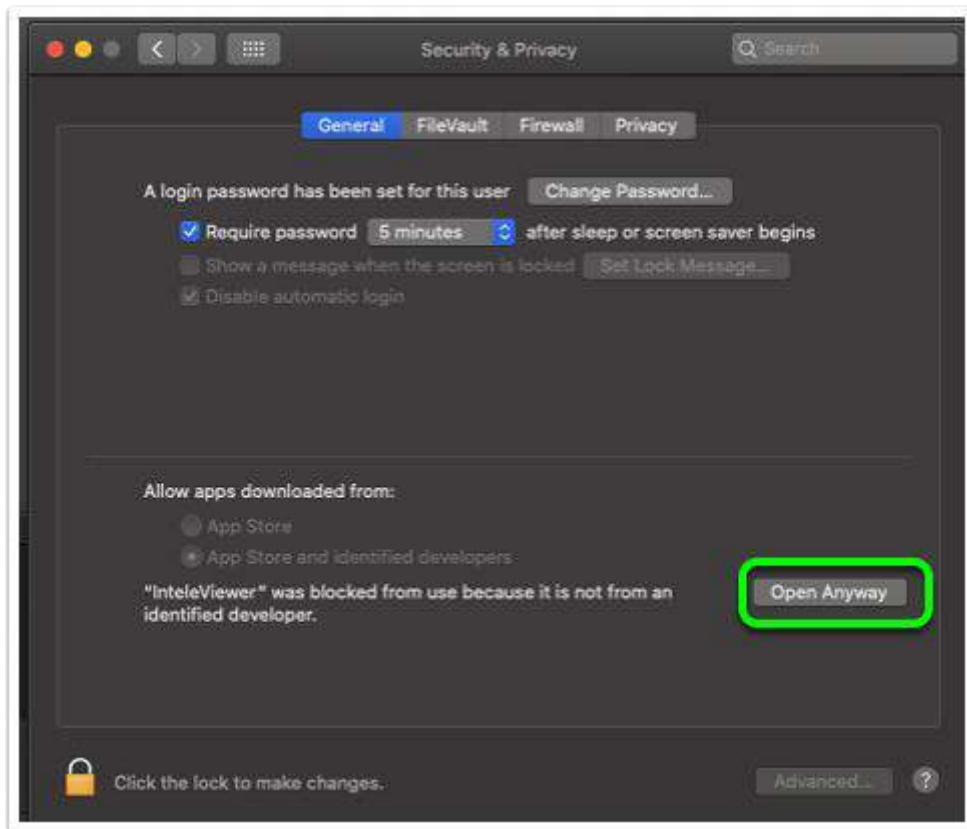


9. Navigate back to Applications and open IntelViewer

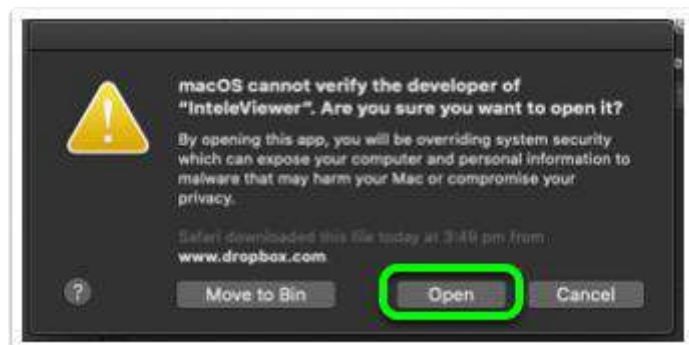

10. If you have not had IntelViewer on this computer before you will need to add the bookmark for PRP, click on **EDIT BOOKMARKS**:



11. The IntelViewer / PACS bookmark for PRP is: <https://pacs.prpimaging.com.au>  
You can give it any name you like but we use **PRP PACS**:

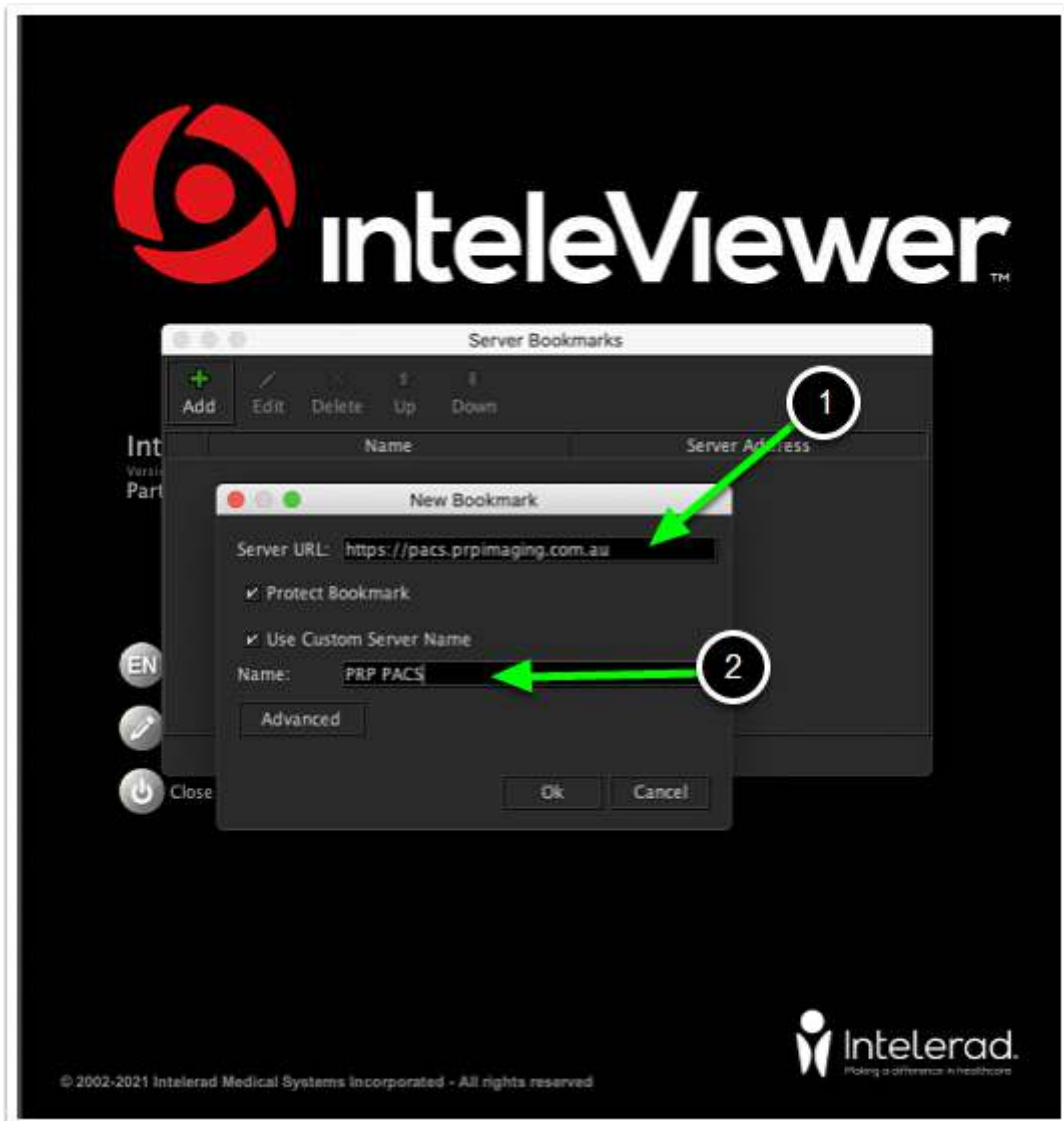

12. You should then be able to login with your PRP IntelViewer username and password.